



# DISCOVER<sup>®</sup>

pregnancy test

at home test  
for the detection  
of pregnancy

## BEFORE YOU BEGIN

*Please read all information in this leaflet before performing the test.*

Do not use past the expiry date stamped on the bottom of the carton. Store in a dry place below 30°C. Do not freeze. Keep out of reach of children. For in-vitro diagnostic use. Not to be taken internally.

## THE DISCOVER<sup>®</sup> PREGNANCY TEST . . .

- ✓ **Easy-to-read results**
- ✓ **Simple to use**  
Place test strip in cup with urine for 10 seconds.
- ✓ **Use any time of day**
- ✓ **Fast**  
Results in as little as 2 minutes.


## HOW THE DISCOVER<sup>®</sup> PREGNANCY TEST WORKS . . .

The DISCOVER<sup>®</sup> Pregnancy Test detects the pregnancy hormone called hCG (human chorionic gonadotropin) in your urine. The amount of pregnancy hormone in your body increases during the first three months of pregnancy.

The DISCOVER<sup>®</sup> Pregnancy Test is so sensitive that there is usually enough hCG to be detected as early as the first day of your missed period.

## INSTRUCTIONS FOR USE

- Collect some of your urine in the cup provided.
- Remove the test strip from the foil wrapper.
- Hold the Dipping End of the strip in the cup of urine for 10 seconds only. **The urine level should not be higher than the maximum level line printed on the strip.**
- Remove the strip from urine and place on a flat non-absorbent surface, with the printed side facing up.
- You should see a pale pink line in the Result Window. This indicates the test is working. The results should be visible in about 2-5 minutes.

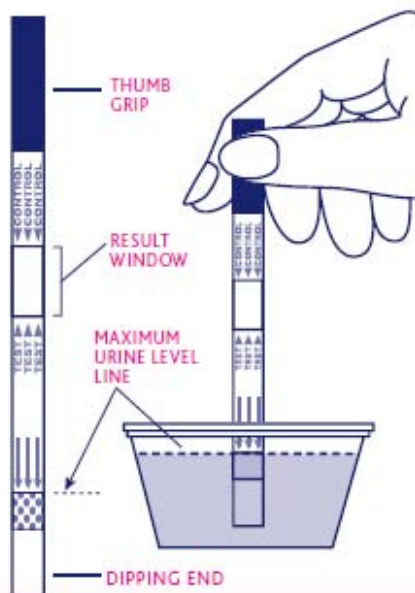

## RESULTS



### Pregnant

The appearance of two pink lines in the Result Window means you are likely to be pregnant. **One line may be lighter than the other, they do not have to match.**



### Not Pregnant

The appearance of one pink line in the Result Window means you are likely to be not pregnant.



### Invalid

If no line appears in the Result Window after 5 minutes, the result is invalid and you should repeat the test with another DISCOVER<sup>®</sup> Pregnancy Test, ensuring instructions are followed carefully.