

Please read this leaflet carefully before performing this test.

DISCOVER ONESTEP In-stream Pregnancy Test is a very simple, fast, one step test which will give you a reliable result as early as **6 DAYS BEFORE THE DAY OF YOUR MISSED PERIOD** (5 days before the day of expected period). DISCOVER ONESTEP In-stream Pregnancy Test contains SENSORFLOW[®], so sensitive it detects **extremely low levels of pregnancy hormone**, up to 6 days before the day of your missed period (five days before day of expected period). In clinical testing, DISCOVER ONESTEP In-stream Pregnancy Test detected hormone levels consistent with pregnancy in 62% of women 6 days before their missed period (5 days before the day of expected period), in 78% of women 5 days before their missed period, in 87% of women 4 days before their missed period, in 98% of women 3 days before their missed period and **over 99% 2 days before the day of their missed period**. It can tell you're pregnant even when your pregnancy hormones are at a very low level. Discover understands how important it is to gain reliable results.

Pregnancy Testing, SENSORFLOW[®] and You

1. Pregnancy hormone levels vary from one woman to the next (Due to factors such as stress, how much water or other liquid you have drunk before testing, cycle length and genetics).
2. Varying levels of pregnancy hormone in your body affect the reliability of the result. Because of this, you need a test that's reliable.
3. Reliability = accuracy + high sensitivity to pregnancy hormones.
4. DISCOVER ONESTEP In-stream Pregnancy Test contains SENSORFLOW[®], so sensitive it detects even extremely low levels of pregnancy hormone, up to 6 days before the day of your missed period (5 days before the day of expected period). It can tell you're pregnant, even when your pregnancy hormone levels are at a very low level.
5. SENSORFLOW[®] allows you to test anytime of day, up to 6 days before the day of your missed period (5 days before the day of expected period).



FIGURE 1

DISCOVER[®] with SENSORFLOW[®]



INSTRUCTIONS

DISCOVER ONESTEP In-stream Pregnancy Test can be used any time of day. There is no need to use first morning urine.

- Remove test stick from the foil wrapper and take off the cap.
- Holding the test stick by the Thumb Grip with the Absorbent Tip pointing downward and the Result Window facing away from your body, place the Absorbent Tip in your urine stream for **5 seconds** only. (You may also collect your urine in a clean, dry cup and immerse the entire Absorbent Tip in the urine for 5 seconds)
- With the Absorbent Tip still pointing downward, replace the Cap* and lay the stick on a flat dry surface with the Result Window facing up.
- You may soon see a pink colour moving across the result window to indicate the test is working.
- The results should be read from 3 to 10 minutes after taking the test. The accuracy cannot be guaranteed **after 10 minutes**.

* Replacement of the Cap is not necessary for the proper functioning of this test. The Cap allows you to save the test in order to show a positive result to your partner or doctor.

READING THE TEST RESULT



PREGNANT

PREGNANT

- 2 Pink lines in the result window. (One line may be lighter than the other, they do not have to match).



NOT PREGNANT

NOT PREGNANT

- 1 Pink line in the result window. (There will be no trace of a second line).